

# Innowalk Pro

The motorised full-body exercise trainer

IN AND OUT  
QUICK-GUIDE



## Pre-check

Pre-check is a check performed to discover wear on important parts. The most important parts are checked daily before use. Other parts are checked for wear once a month.



For more tips on how to execute the pre-checks, scan the QR code and examine the Pre-check instructions given.

## Preparation before transfer

- Turn on the device
- Adjust the device to the user/patient in accordance to personal settings
- Place the Innowalk in sitting position
- Swing away and lower the leg supports (A)
- Open all straps, chest and hip support, release the guide-straps


### If swivel seat is used for more independent transfer

- Position foot plate on the entrance side in lowest position in the walking cycle
- Turn swivel seat to preferred entrance side (B)
- Adjust seat to preferred height on the remote control for easy transfer from ex. Wheelchair

### If hoist is used for transfer

Use the tilt function to enable access of the legs of the manual hoist under the Innowalk Pro (not needed if a ceiling hoist is used)

A

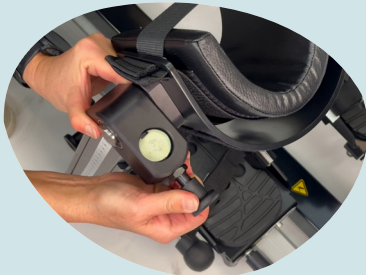

Pull out the handle (1) lift up (2) and swing away the leg support.



Pull out the handle (1) and lower down the leg support.

B



Pull down the handle (1) turn the seat.



## Transfer the user into the Innowalk Pro

- Transfer the user into the seat and support the person until they are secured by the chest support
- Secure the chest support
- If swivel seat is used Increase seating height and turn the seat to front position (C)
- Secure straps over the feet (D)
- Secure the leg supports (E)
- Secure the guide-straps (F) and tighten to defined tension
- Move the user from sitting to defined standing position
- Secure the hip belt in standing (G)



## Support Equipment

- Attach shoulder straps (H)
- Attach and adjust position of the tray (I and J)
- Attach and adjust position of the arm movement handles (K)



## Transfer the user out of the Innowalk Pro

- Remove support equipment
- Release the hip belt
- Lower to sitting position
- Release the straps over the feet
- Release the guide-straps and swing away and lower the leg supports

### **If swivel seat is used for more independent transfer**


- Turn the swivel seat to preferred side
- Lower the seat for preferred transfer position
- Release the chest support
- Transfer the person out of the device


### **If hoist is used for transfer**

- Attach the slings to the hoist
- Release the chest support
- Transfer the person out of the device

# Innowalk Pro

**For more information,  
please contact us.**

 +44 7885 478 000

 [madeformovement.com/gb](https://madeformovement.com/gb)

Subject to printing errors / MfM-ENG-0124 / © 2024 Made for Movement AS. All rights reserved.