

INNOWALK

Enables activity every day

IN AND OUT
QUICK-GUIDE






Pre-check

Pre-check is a check performed to discover wear on important parts. The most important parts are checked daily before use. Other parts are checked for wear once a month.



For more tips on how to execute the pre-checks, scan the QR code and examine the Pre-check instructions given.

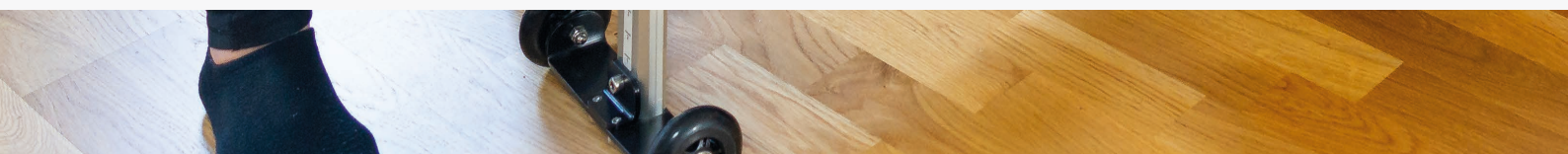

Preparation before transfer

- Turn on the device
- Place the Innowalk in sitting position
- Swing away and lower the leg supports (A)
- Open all straps, chest and hip support, release the guide-straps


If swivel seat is used for more independent transfer

- Position foot plate on the entrance side in lowest position in the walking cycle
- Turn swivel seat to preferred entrance side (B)
- Adjust seat to preferred height on the remote control for easy transfer from ex. Wheelchair (Size M and L)

If hoist is used for transfer

Use the tilt function to enable access for the «legs» of the manual personell lifter under the Innowalk (not needed if a ceiling hoist is used).

A

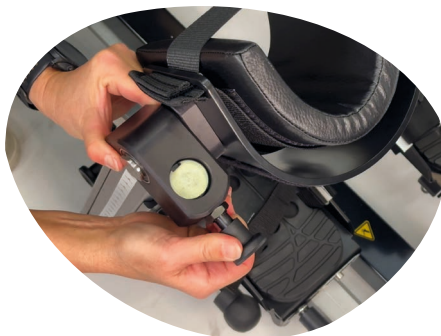

Pull out the handle (1) lift up (2) and swing away the leg support.



Pull out the handle (1) and lower down the leg support.

B



Pull down the handle (1) turn the seat.

Transfer the user into the Innowalk

- Transfer the user into the seat and support the person until they are secured by the chest support
- Secure the chest support and attach the hip belt loosely
- If swivel seat is used Increase seating height and turn the seat to front position (C)
- Secure straps over the feet (D)
- Secure the leg supports (E)
- Secure the guide-straps (F) and tighten to defined length
- Move the user from sitting to defined standing position
- Secure the hip belt in standing (G)



Transfer the user out of the Innowalk

- Release the hip belt
- Lower to sitting position
- Release the straps over the feet
- Release the guide-straps and swing away and lower the leg supports

If swivel seat is used for more independent transfer

- Turn the swivel seat to preferred side
- Lower the seat for preferred transfer position
- Release the chest support
- Transfer the person out of the device


If hoist is used for transfer

- Attach the slings to the hoist
- Release the chest support
- Transfer the person out of the device

INNOWALK

**Feel free to reach out to your
Made for Movement representative if
you require any assistance with the
utilization of the Innowalk.**

 +47 35 50 51 20

 madeformovement.com/no

Subject to printing errors / MfM-ENG-0124 / © 2024 Made for Movement AS. All rights reserved.